

USE OF THE

Bolin Chest Seal

Valved Chest Dressing




Clean & dry area around wound before application.



Tear open the package by using the indicated tear notches remove the product. Grab the product by the large tab to keep the product sterile. Then follow the instructions below.

1


Using the provided pad, wipe away any bodily fluids from the chest wound. Holding the product at the large tab, peel the clear plastic backing away from the sticky hydrogel base.

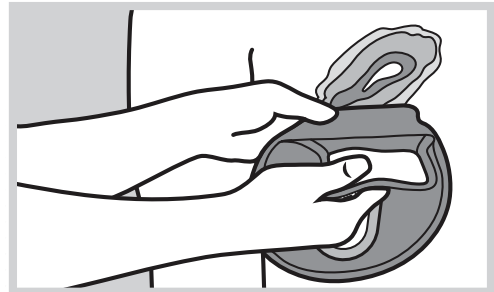

2

Hold the Bolin Chest Seal over the wound and place it on the chest, centering the three valves over the wound area. Air will begin to exit through the one-way valves. Press the seal down onto the chest to ensure a complete seal.

If one of the valves becomes blocked, lift the Bolin Chest Seal and reposition it so that one of the other valves is centered over the wound.

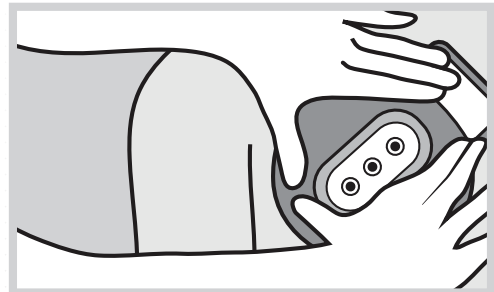

Continue to monitor the injured for signs of a tension pneumothorax. Follow local protocols for treatment of a tension pneumothorax when using the Bolin Chest Seal.



THIS PRODUCT IS SINGLE USE ONLY. DO NOT REUSE.

This product is a medical device that may require specific training regarding proper use. Please follow any local guidance or governing authority regarding the use of this device.