

USE OF THE

# H\*VENT

## Vented Chest Dressing




Clean & dry area around wound before application.



Tear open the package by using the indicated tear notches remove the product. Grab the product by the large tab to keep the product sterile. Then follow the instructions below.


**1**

Ensure a firm grip on the seal pull-tab with the thumb against the white side. If the tab has peeled away from the base of the seal, do not pull on the clear base. Slowly peel away the clear liner to expose the hydrogel.

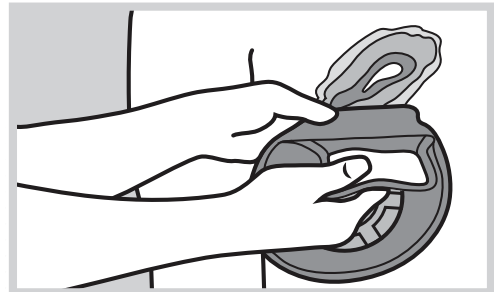

**2**

Hold the H\*VENT over the wound and place it on the chest, centering the vents over the wound area. Air will begin to exit through the vents. Press the seal down into the chest to ensure a complete seal.

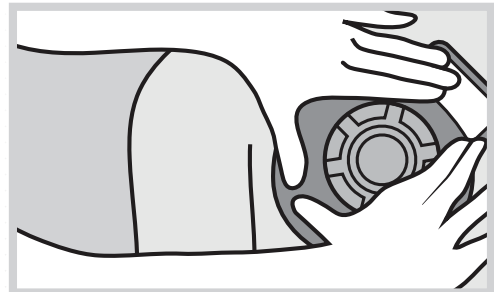

\*

Continue to monitor the injured for signs of a tension pneumothorax. Follow local protocols for treatment of a tension pneumothorax when using the H\*VENT.



**THIS PRODUCT IS SINGLE USE ONLY. DO NOT REUSE.**

*This product is a medical device that may require specific training regarding proper use. Please follow any local guidance or governing authority regarding the use of this device.*