

USE OF THE

SWAT-T™


STRETCH, WRAP AND TUCK TOURNIQUET



Open the packaging to the tourniquet by either tearing the package open at the tear notches or by grabbing the ends of the package and twisting the product, creating a tear in the middle. Remove the product from the package and set the tourniquet application band aside.


1

Wrap the SWAT-T above the wound area (or over the wound area if being used as a compression dressing). Wrap once around the extremity to ensure the tourniquet is secure against the skin.

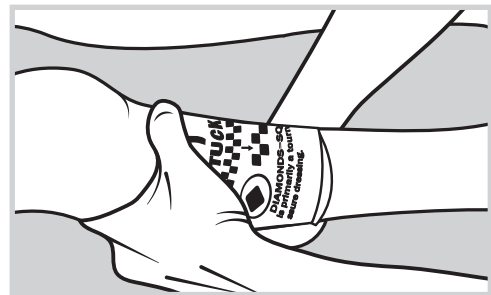

2

Pull the SWAT-T tight against the extremity until the graphic diamonds on the tourniquet become squares. This will show that enough force is being applied to stop severe bleeding.

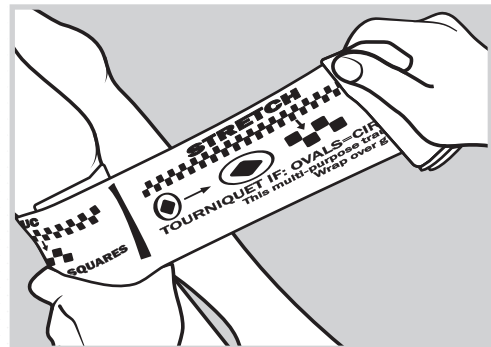


3

Continue to wrap the SWAT-T around the extremity, maintaining the compression by continuing to pull the tourniquet while wrapping.

4

Once near the end of the tourniquet, hold the last wrap up to create a loop and tuck the end of the tourniquet under this loop. The SWAT-T will grip onto itself and remain in place. Frequently check any style tourniquet for continued occlusion as loosening can occur after application.

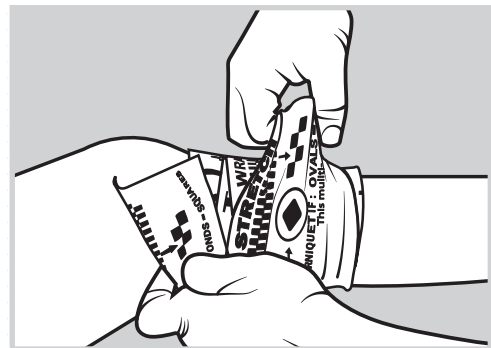

THIS PRODUCT IS SINGLE USE ONLY. DO NOT REUSE.

This product is a medical device that may require specific training regarding proper use. Please follow any local guidance or governing authority regarding the use of this device.