

USE OF THE H&H

# Wound Seal

OCCLUSIVE DRESSING



Clean & dry area around wound before application.



Tear open the package by using one of the indicated tear notches and remove the product. Grab the product by the large tab to keep the product sterile. Then follow the instructions below.


**1**

Holding the product at the large tab, peel the clear plastic backing away from the sticky hydrogel base.




**2**

Hold the Wound Seal over the wound and place it on the wound, centering the seal over the wound area. You can watch and monitor the wound through the hydrogel.

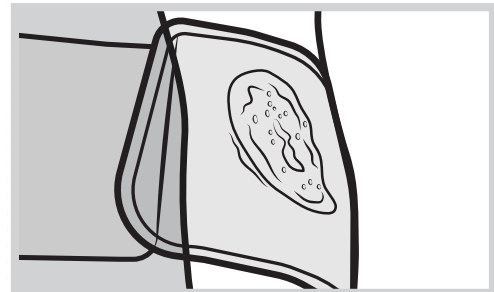

**3**

Smooth out the hydrogel to ensure a firm seal against the skin. If you need to adjust or move the seal, use the large tab at one end to pull the seal away from the skin.

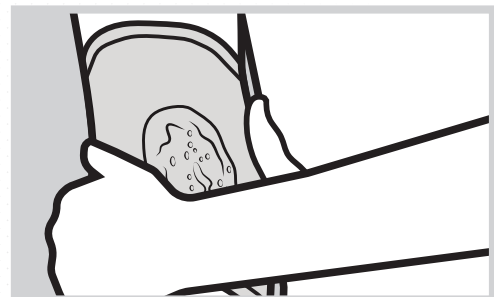

**THIS PRODUCT IS SINGLE USE ONLY. DO NOT REUSE.**

*This product is a medical device that may require specific training regarding proper use. Please follow any local guidance or governing authority regarding the use of this device.*