

USE OF THE

Dual Seal

Chest Seal Two-Pack




Clean & dry area around wound before application.



Inspect the foil pouch to make sure the packaging has not been compromised. Tear open the package by using one of the indicated tear notches and remove the product (2 seals included). Then follow the instructions below.


1

Remove the seal from the protective backing and apply to the wound with the hydrogel adhesive applied against the wound and skin.

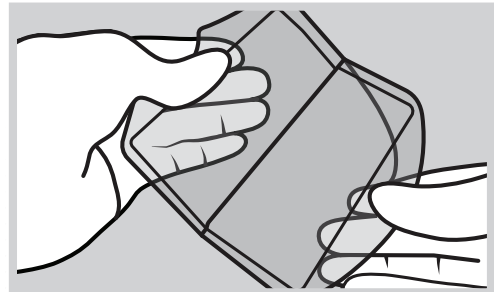

2

Smooth the seal over the wound and apply pressure to form a protective seal around the wound site. Monitor the wound area until seen by medical professionals.

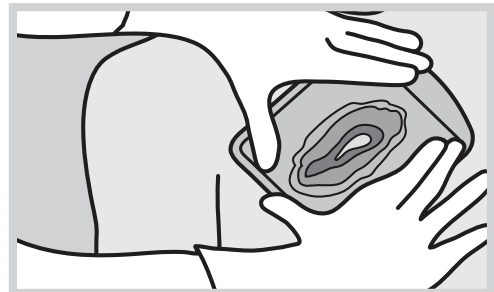

THIS PRODUCT IS SINGLE USE ONLY. DO NOT REUSE.

This product is a medical device that may require specific training regarding proper use. Please follow any local guidance or governing authority regarding the use of this device.