

USE OF THE H&H

Combat Eye Shield




Using the included gauze, wipe around the injury site, removing bodily fluids or debris from around the eye. Do not wipe the eye itself and do not place the gauze or any item on the eye.



Tear open the package by using one of the indicated tear notches and remove the product from the package. Remove the eye shield from the protective wrapper. Then follow the instructions below.


1

Peel back the clear protective sheet away from the hydrogel side of the seal. Make sure the aluminum eye shield is firmly embedded in the seal itself.

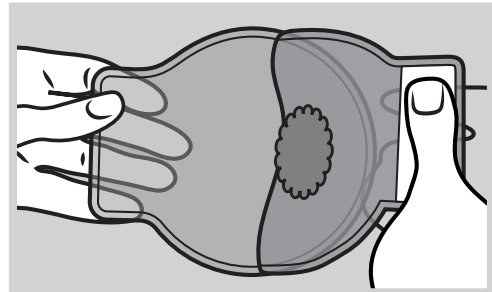


2

Position the eye shield directly over the injured eye and smooth the hydrogel disc onto the skin. Try to remove any wrinkles or folds in the disc to ensure a proper seal.

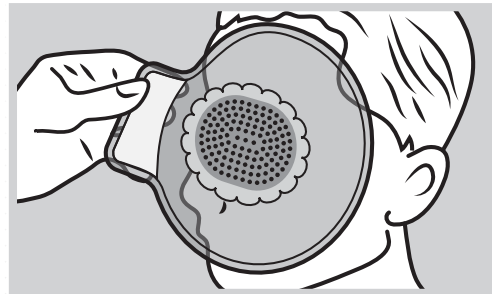

3

If the hydrogel disc is too large for the injured person's face or may block the nose or mouth, trim away the portions of the disc that may cause issues.

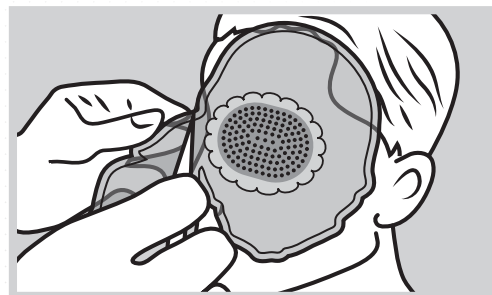

Frequently check the eye shield to make sure it has not moved or needs to be repositioned.



THIS PRODUCT IS SINGLE USE ONLY. DO NOT REUSE.

This product is a medical device that may require specific training regarding proper use. Please follow any local guidance or governing authority regarding the use of this device.