

USE OF THE

THE ECD


Emergency Compression Dressing



Tear open the package by using one of the indicated tear notches and remove the product. Grab the product by the elastic to keep the pad side sterile. Then follow the instructions below.

1

Apply the bandage with the sterile pad down over the wound and the elastic wrap side up. Take the elastic wrap and pull it around the bandage, securing the short elastic end to the wrap.

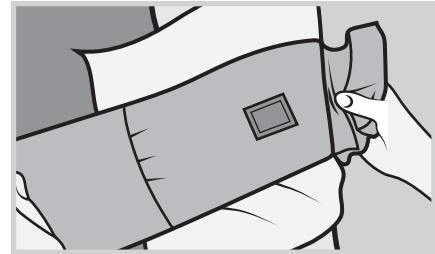


2

Once the bandage is secure, begin wrapping the elastic around the wound, keeping tension on the elastic. This will add pressure around the wound and help reduce bleeding. Continue to wrap the elastic until you reach the end.



3

Once finished wrapping all of the elastic, secure the end to the wrap with the hook material to the wrap or use the barrette to secure the end to the wrap.

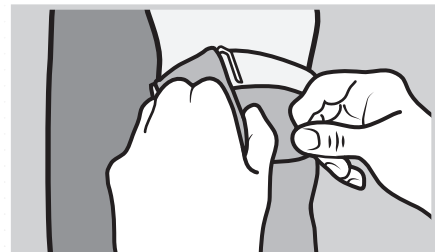

THIS PRODUCT IS SINGLE USE ONLY. DO NOT REUSE.

This product is a medical device that may require specific training regarding proper use. Please follow any local guidance or governing authority regarding the use of this device.