

WOUND PACKING WITH

H&H Gauzes




Wear protective gear such as gloves before using this product as you will come in contact with blood and/or bodily fluids.



To start, tear open the package by using one of the tear notches on the package and remove the product. Then follow the instructions below.


1

Find the end of the gauze roll by either bending the roll in the middle or pulling on the top. Take the end and pull about 6-8" of gauze from the roll. Hold the end in one hand and the rest of the roll in your other hand.

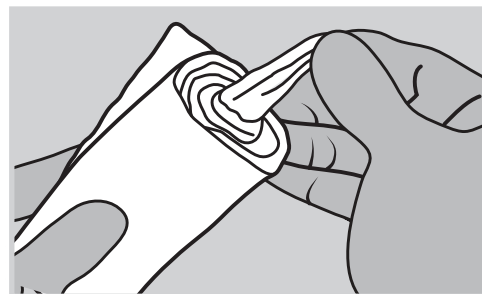

2

Using one or two fingers, push the end of the gauze roll deep into the wound. Push the end of the gauze all the way down into the wound until you reach the end of the wound. Bring your fingers out of the wound and pull more of the gauze from the roll.

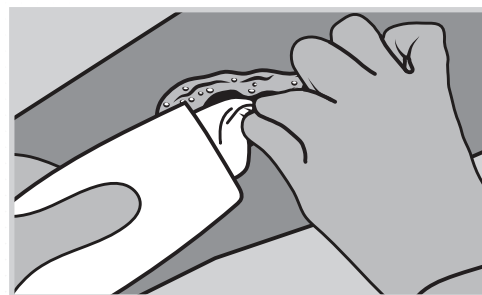


3

Continue pushing gauze into the wound, filling the entire wound cavity with gauze. Use your fingers to push the gauze firmly down into the wound to apply pressure. Once the gauze is completely packed into the wound, place the remainder of the gauze roll (if any) on top of the wound and apply firm pressure over the wound.

4

Hold this pressure for three minutes. Do not release pressure to check the wound during this time. After three minutes, apply a compression bandage or securing bandage to keep the gauze in place.

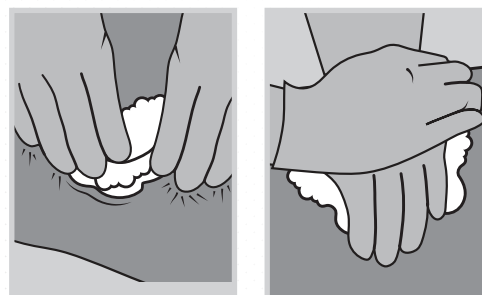

THIS PRODUCT IS SINGLE USE ONLY. DO NOT REUSE.

This product is a medical device that may require specific training regarding proper use. Please follow any local guidance or governing authority regarding the use of this device.